

cobas bge link software

Manages cobas® blood gas instruments and data remotely

cobas bge link software saves you time and streamlines the management of blood gas testing, giving you information and control, for confidence and efficiency.



Information when you need it

- Use real-time screen-sharing from your computer to identify which analyzers are ready or need attention.
- View the status of all consumables (reagents, controls, sensors).

Remote control

- Access any **cobas** blood gas analyzer remotely. Send commands to run calibration, QC, wash, and device and parameter lock/unlock.
- Review patient results and QC data as well as calibration information.

Confidence

- Enter pertinent information, such as vent settings, and document actions taken on critical results prior to transmitting patients' reports to the LIS or HIS system.
- Manage operators' access without leaving your office.
- Monitor blood gas testing through review of maintenance, operator and QC reports.

Efficiency

- Use web validation from any computer connected to the hospital intranet to review patient and QC results.
- Take action using remote control instead of being at the analyzer.

IT system compatibility

- Supported in 64-bit operating systems
- Supported in VMware virtualized environments (ESX 4.0 or higher)




cobas®

Life needs answers

cobas bge link software

Provides a comprehensive audit trail of analyzers, patient and QC results, maintenance activities, and error logs

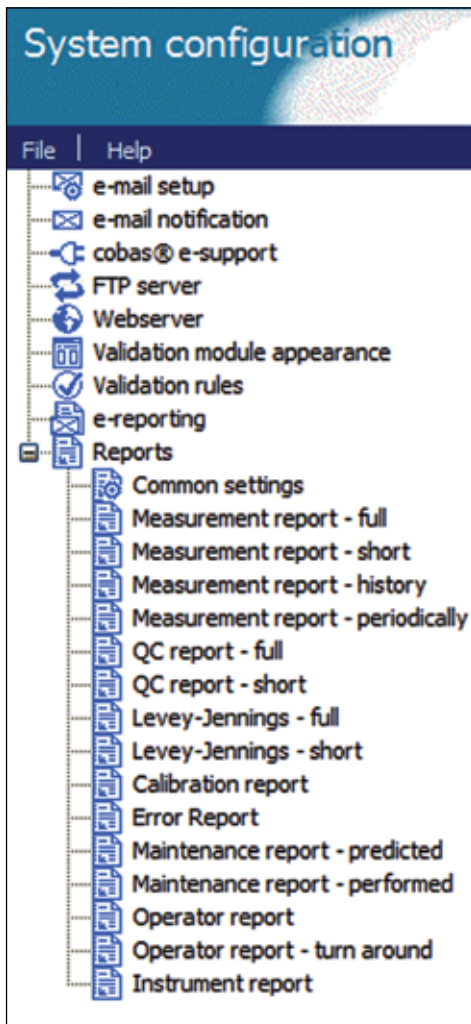

Compliance

- Verify that operators have appropriate analyzer access.
- Validate results and QC from the web at any time.
- Generate pre-defined and customizable reports to help you meet regulatory and compliance requirements.
- Have customized reports routinely emailed to you.

Reports available with cobas bge link software include:

- Multiple patient reports
- QC results
- Instrument, calibration and error reports, including information explaining why a measurement did not occur
- Performed and predicted maintenance reports
- Customizable reports based on desired date and frequencies

To find out more, contact your Roche Diagnostics POC Account Manager, Blood Gas Specialist, or call 1-800-428-5076.

COBAS, COBAS B, COBAS BGE LINK, and LIFE NEEDS ANSWERS are trademarks of Roche. All other product names and trademarks are the property of their respective owners.

© 2012 Roche Diagnostics. All rights reserved. 340-51495-0712

Roche Diagnostics
9115 Hague Road
Indianapolis, IN 46256
www.poc.roche.com